REAL TIME – AVIAN INFLUENZA H5N1 © 2008 Genekam Biotechnology AG MADE IN GERMANY

Lot-No. Ref. FR029

MANUAL Real time

Expiry time: 1 year STORE AT -20°C

100 Tests (Ready to use PCR kit)

Real time - Avian Influenza H5N1

-Only for in vitro use--Only for veterinary use--Only for research use (human)-To be used by a technical person-

Principle and use:

This amplification kit has been manufactured by *Genekam Biotechnology AG*, Germany to detect avian influenza virus H5N1 in real time PCR.

Real time PCR is based on fluorogenic dyes. In our kit we use 2 dyes, they are 6-Carboxy tetramethyl rhodamine (quencher) and Carboxy-fluorescein (reporter). Up to 36 Ct should be taken positive. Value between 36-40 Ct should be taken as marginal positive (doubtful).

This kit needs DNA which can be isolated from nasal swabs, blood, urine, respiratory swabs, lung tissue, faecal samples, blood serum, tissue and any body fluid. Kindly use good methods to isolate the DNA.

Safety precautions should be taken as avian influenza virus is infectious for human beings and animals. Always clean your hands before the test use and clean the hands after the test. Wash your face after the test, if possible. Desinfect your working place.

IMPORTANT: we added cotton or sponge in the lid of container of the kit, to avoid damage during transportation. Please remove this cotton or sponge from the lid of each container before storage.

Composition:

It contains the following:

- Tube A (2 tubes)
- Tube B (2 tubes)
- Tube Y (1 tube)
- Positive (+Ve) Control (tube D1) (1 tube). This tube must be stored at -20°C. The user can also make his own positive control, if this positive control is not funtioning or the quantity is not sufficient.
- Negative (-Ve) Control (tube D2) (1 tube)

Please check them before you start. Please store them at -20°C and dark.


Equipment needed:

- Laboratory centrifuge
- microtubes (0.2ml)
- Pipette-tips with and without filter (10-100µl & 1-10µl)
- Pipettes (quality pipettes)
- Paper
- Pen
- Vortexer
- 96 well microplates for PCR
- Real time machine


Procedure:

STEP A


- 1. Kindly thaw one tube A, B, Y, D1 and D2. After thaw, kindly put the tubes at 4° C (as it is better). If the kit is not in use, store them at -20° C.
- 2. Mark your microtubes with a sample number, +ve Control and -ve Control. You can use 96 well microplate instead of tubes.

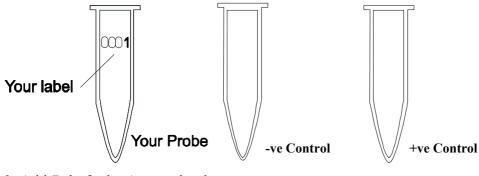


3. Add 7µl of tube A to each tube.

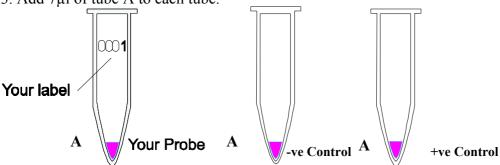

4. Add 10µl of B to each micro tube. Avoid to touch the wall of the microtubes.

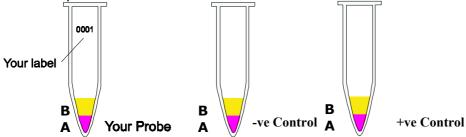


5. Add 1µl of Y to each tube (avoid to touch the wall of the microtubes).

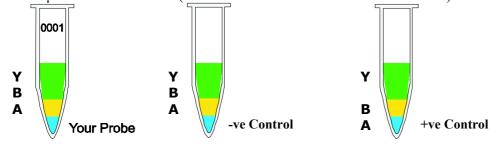


TIP: Add $7\mu IA + 10\mu IB + 1\mu IY = 18\mu I$ per reaction. In case you want to run 10 reactions i.e. you need total $180\mu I$, therefore you should mix $70\mu I$ of $A + 100\mu I$ of $B + 10\mu I$ of $Y = 180\mu I$ from which you can take $18\mu I$ and add to each tube. This way you save time and hardware.


6. Add 2µl of your RNA with sterile pipette-tip with filter to each micro tube according to your label except +Ve and -Ve (Avoid touching the wall). Use every time a new pipette tip (for each sample)! Mix it.

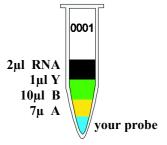

7. Use new pipette tip with filter. Add $2\mu l$ of tube D1. This is the positive control supplied with our kit. Mix it.

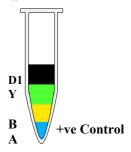


8. Use a new pipette tip. Add 2µl of –Ve (tube D2) to –Ve Control (don't touch the wall). Mix it.

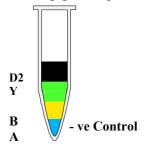

9. Centrifuge all tubes for 20 sec. for 8000 rpm (this is not necessary but it is better).

10. Run the program of your thermocycler as followings: Kindly check whether you have added everything correctly as the level of the volume of each micro tube must be almost the same.

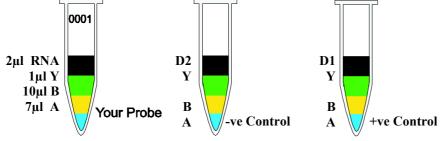

You must use quencher and reporter dye to setup your software (see FAQ) and run the following program:

1. 60 minutes at 42°C 10 minutes at 70°C

2. 15 seconds at 95°C x 40 cycles 60 seconds at 58°C

Before you start the PCR program, kindly check whether tubes are closed properly. **Microtubes must be in contact with metal block** (important!). There should be no air or lose contact with metal block of thermocycler.

11. After step 10 is finished take out the microtubes.

STEP B

Once the program will be finished one can see the graphics. The negative control should run along with the bottom and positive control must give a curve in the software graphics. Use your software to analyse the results.

If you should find any mistakes, please let us know. Thank you.

Suggestion:

This manual has been written specifically for beginners, hence persons with experience in PCR must use their experience to keep each step as small as possible e.g. you should calculate the amount of the needed chemicals, before starting with testing.

Last update: 06-06-2011

v1.3

Genekam Biotechnology AG
Dammstr. 31-33
47119 Duisburg
Germany
Tel. (+49) 203 / 555858-31,-32,-33
Fax (+49) 203 / 35 82 99
anfrage@genekam.de

http://www.genekam.de

FAQ:

1) Q: I cannot find quencher and reporter dye in my software:

A: Many software has got the words: FAM (as reporter) and TAM (as quencher). Therefore select both in your software.

If your machines has only one word (for some machines only use the word FAM) you

should select this one.